

- [Home](#)
- [Current Edition](#)
- [Magazine Library](#)
- [Bite iPad app](#)
- [Advertise](#)
- [Subscribe](#)
- [Contact Us](#)

# Bite magazine


- [Dental News](#)
- [Design + Fitout](#)
- [Gear](#)
  - [New Products](#)
  - [Product Guides](#)
    - [Dental Units & Chairs](#)
    - [Digital Imaging](#)
    - [Education](#)
    - [Ergonomic design](#)
    - [Finance](#)
    - [Handpieces](#)
    - [Implants](#)
    - [Infection Control](#)
    - [Intra-oral cameras & magnification](#)
    - [Practice management software](#)
    - [Web & Email Solutions](#)
  - [Tool Reviews](#)
- [Passions](#)
- [Your Business](#)
- [Your World](#)

You are here: [Home](#) » [Gear](#) » [Tool Reviews](#) » **Tools of the trade: Dyloc Sutures**



## Tools of the trade: Dyloc Sutures

[admin](#) | February 22, 2013 | [Reply](#)

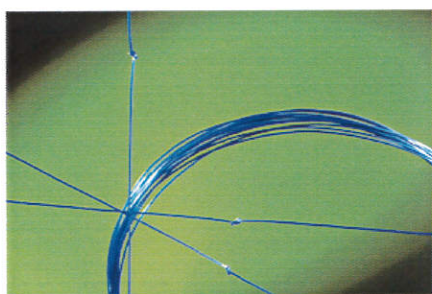

Dyloc sutures are the best available, says Dr Azmi

by Dr Karim Azmi, Lynwood Dental, Lynwood, WA

We were approached by a sales rep from Dynek who sent us samples of these sutures. I do a lot of surgery, and I can say these are fantastic—the best sutures I've ever used.

### What's good about it

The Dyloc sutures are monofilament polyether so the whole thing is only one string. For normal surgical procedures, I prefer 4/0 reverse cutting while for cosmetic procedures, implant surgery or anything at the front, I use 6/0 premium reverse cutting.

During healing it keeps everything very clean as food doesn't get attached to it. Healing happens very fast as well. One of my patients had a suture left in for three weeks as they went away on holiday. When they returned, I couldn't even see the suture as the tissue had grown all around it. I had to dig to remove it. I would say that 10 days would be ample time before removing them.

It locks very well and I rarely have any slippage. It's a purple colour so I can see it in any situation, even if the wound is bleeding. The needle is extremely sharp and penetrates easily.

I love that it's Aussie made and in my opinion, it beats anything else I've ever used. On top of that, they're way cheaper than other brands.

### What's not so good

I can't really find anything wrong with them. I've been using them for the past few months and I've been very impressed. Even their customer service is very good. Although they are Adelaide based, we placed an order and a day later it was here in Perth.

### Where did you get it

[Dynek Sutures](#)

Bite Magazine and website is published by [Engage Media](#). All material is protected by copyright and may not be reproduced in any form without prior written permission.

Bite magazine and its associated website is published by [Engage Media](#). All material is protected by copyright and may not be reproduced in any form without prior written permission.

Share this:

Facebook

LinkedIn

Twitter

Google

Pinterest

Email

More

Like this:



Be the first to like this.

### Related posts:

1. [Tools of the Trade Review: Waterlase MD](#)
2. [Tools of the Trade: SIRO Torque L+](#)
3. [Tools of the Trade: Waterlase Dentistry](#)
4. [Tools of trade: Roeko Flexi Dam](#)

**Tags:** [dental product reviews](#), [Dyloc sutures](#), [tools of the trade](#)

**Category:** [Tool Reviews](#)

### Leave a Reply

Name ( required )

Email ( required; will not be published )

Website

Notify me of followup comments via e-mail

Notify me of new posts by email.

« [ADA campaigns against stupidity](#)  
[Ancient diet better for teeth](#) »